


Atrix-C[®]


Cervical Allograft Interbody Spacer

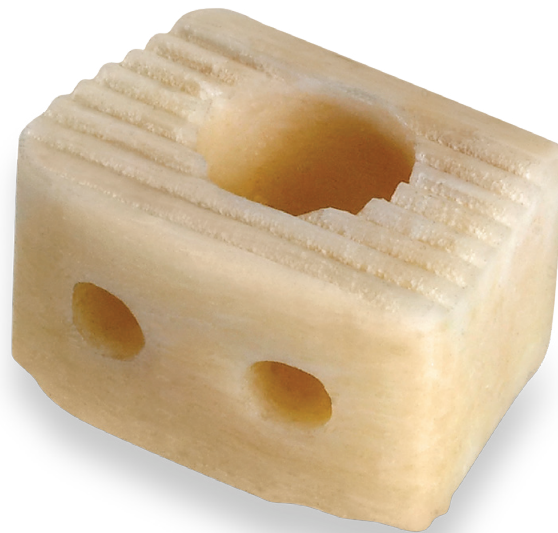

SURGICAL TECHNIQUE

Atrix-C[®]

Cervical Allograft Interbody Spacer






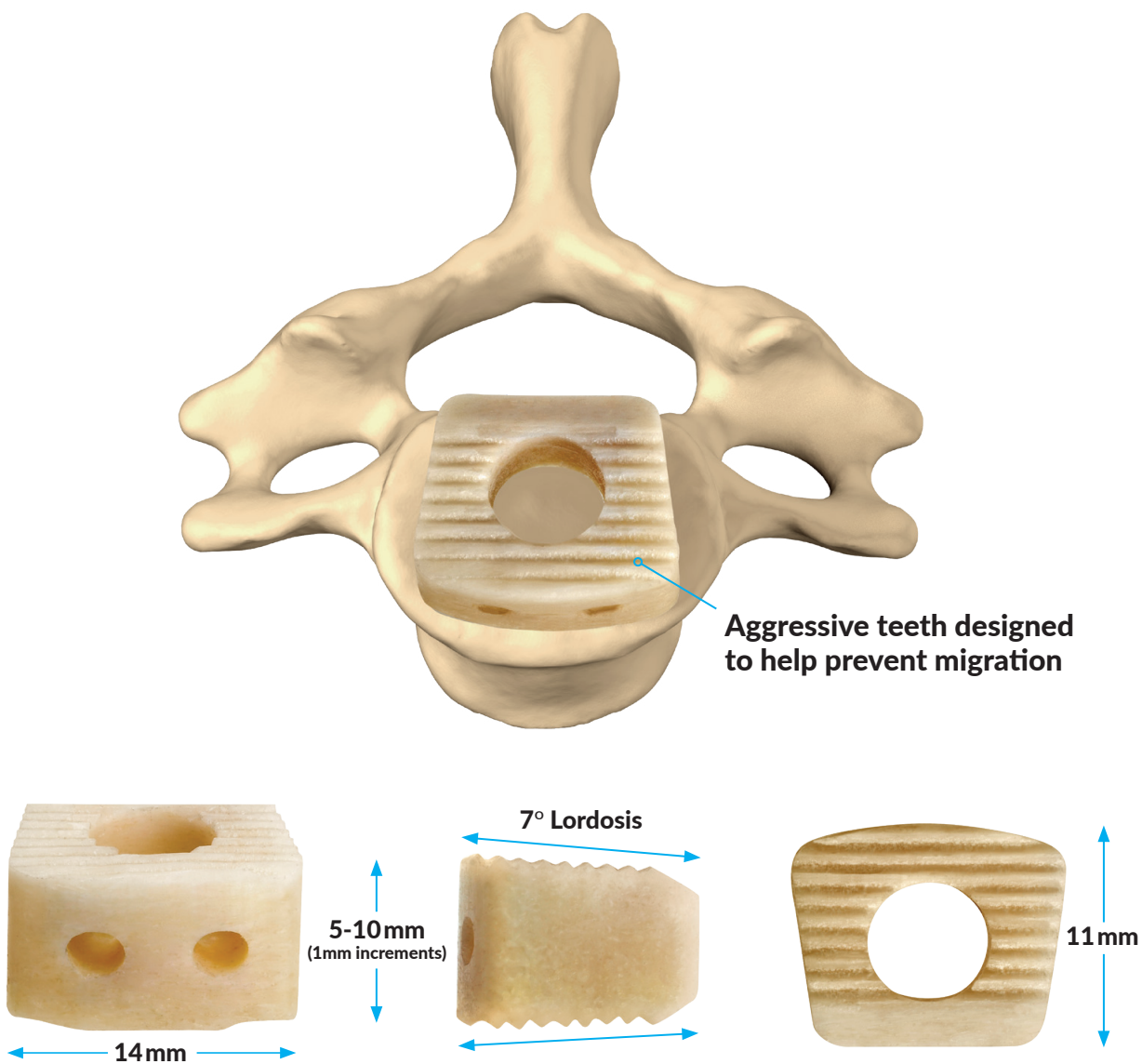


Atrix-C[™] Cervical Allograft Interbody Spacers

Xtant Product Code	Description
X094-1005	Atrix-C Cervical Allograft Spacer, Cortical, 5mm X 7°
X094-1006	Atrix-C Cervical Allograft Spacer, Cortical, 6mm X 7°
X094-1007	Atrix-C Cervical Allograft Spacer, Cortical, 7mm X 7°
X094-1008	Atrix-C Cervical Allograft Spacer, Cortical, 8mm X 7°
X094-1009	Atrix-C Cervical Allograft Spacer, Cortical, 9mm X 7°
X094-1010	Atrix-C Cervical Allograft Spacer, Cortical, 10mm X 7°

Atrix-C® Inserter Assembly

- 1: Line up the laser marks on the distal end of the inserter shaft.
- 2: Place the inserter tip into the inserter.
- 3: (A) Slide the thread retaining shield over the inserter tip, (B) and turn clockwise to lock in place.
- 4: Rotate the knob counter-clockwise ('release' laser marking direction) to open the inserter tip arms.
- 5: Place allograft interbody spacer onto the inserter tip.
- 6: Rotate the knob clockwise ('tighten' laser marking direction) to close inserter tip arms.
- 7: To release the allograft interbody spacer, rotate the knob counter-clockwise ('release' laser marking direction) to open the inserter tip arms.
- 8: The inserter may now be pulled out, leaving the allograft interbody spacer in place.

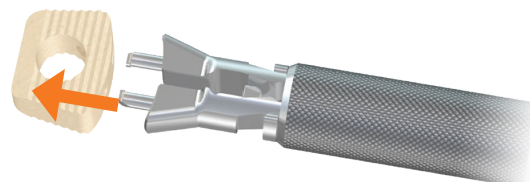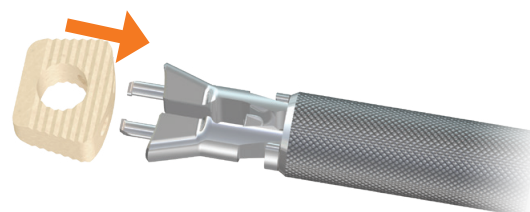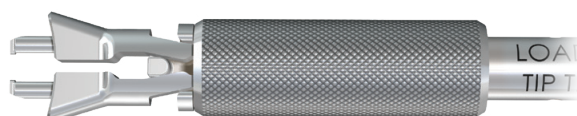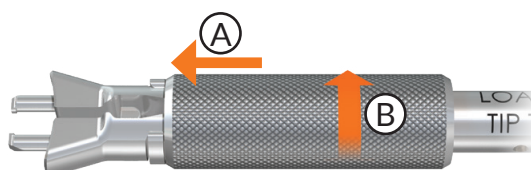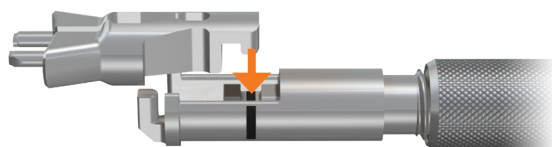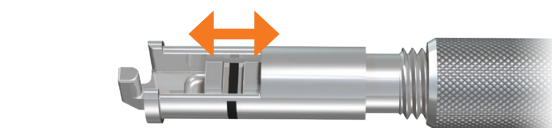

Note: When either releasing or tightening the grip on allograft interbody spacer, always rotate the knob as far as possible in the appropriate direction. For tightening, this ensures an adequate grip. For releasing, this ensures the hook arms are clear of the allograft inserter holes.



☎ 888.886.9354
✉ cs@xtantmedical.com
🌐 xtantmedical.com
📍 664 Cruiser Ln,
Belgrade, MT 59714

INDICATIONS: See Package Insert for a more complete listing of indications, contraindications, warnings, precautions, and other important information.

LIMITED WARRANTY and DISCLAIMER: Xtant Medical products have a limited warranty against defects and workmanship and materials. Any other express or implied warranties, including warranties of merchantability or fitness, are disclaimed.

WARNING: In the USA, this product has labeling limitations. See package insert for complete information. CAUTION: USA Law restricts these devices to sale by or on the order of a physician.

Atrix-C is a registered trademark of Xtant Medical.

© Xtant Medical. All Rights Reserved.

STG-000026 (A) 6/23